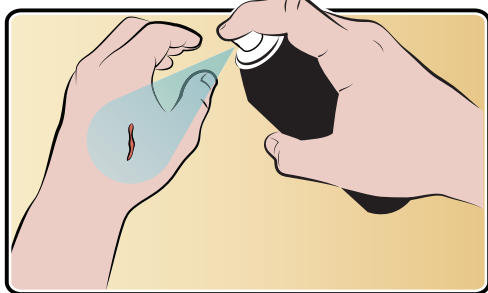
# Wound and Eye Wash Spray

TD-1456, Aurena Wound and Eye Wash

# Wound + Eye Wash Spray

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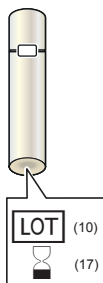
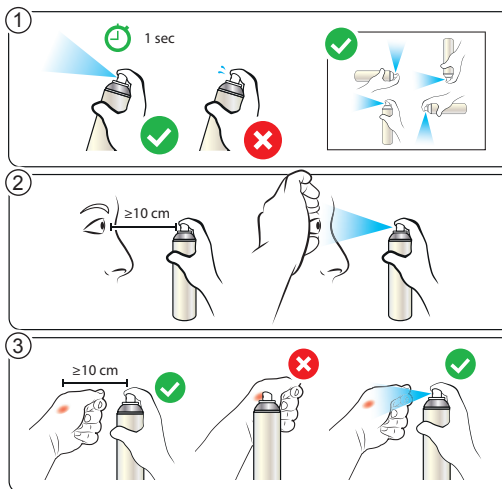
405

**(en) Intended use:**

Sterile isotonic saline solution for rinsing/cleansing of eyes and wounds.

**Target user:**

For all ages. To be used for self-care, at institutions or workplaces without technical experience or training. Can be used with contact lenses.

**Storage:**

Store between 15°C and 30°C, for comfortable use.

Storage/transport temperature: -20°C to 40°C.

**Disposal:**

Follow local regulations for safe disposal of pressurized containers.

**Precautions:**

- Avoid direct contact between the nozzle and the wound/eye.
- Seek medical attention if condition persists or gets worse.
- If any serious incident occurs in relation to the device, report to the local competent authority and the manufacturer.

**Warning**

- Pressurised container. May burst if heated.
- Do not pierce or burn, even after use.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not expose to temperatures exceeding 50°C/122°F.
- Protect from sunlight.
- Keep out of reach of children.

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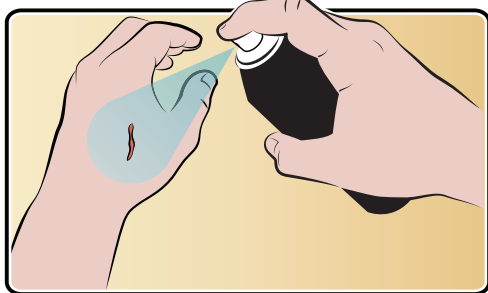
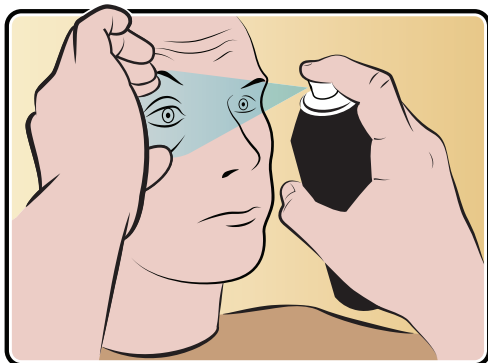
# Wound and Eye Wash Spray

## WOUND AND EYE WASH SPRAY

Phosphate Buffered Sodium Chloride Solution (0.9% NaCl)

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Fosfatbuffrad Koksaltlösning (0,9% NaCl)

Distributör/Distributör:

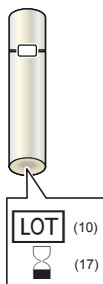
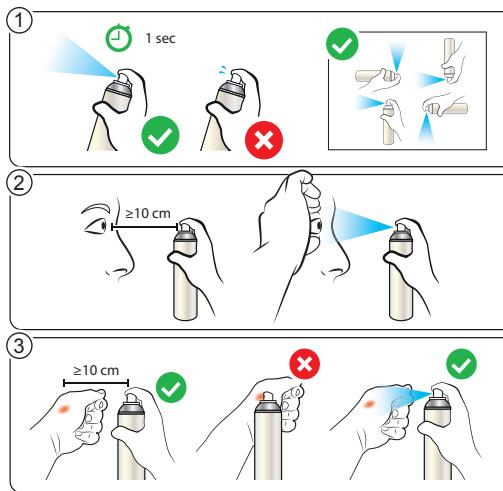
XXXX

(01) XXXXXXXXXXXXXC

V5. 2022-10-18AP

3

STERILE R

CE 2862

Aurena Laboratories AB  
Fjärriksvägen 22,  
SE-653 50 Karlstad  
Sweden/Sverige

REF 2002-2

MD

250 ml

405

### (en) Intended use:

Sterile isotonic saline solution for rinsing/cleansing of eyes and wounds.

### Target user:

For all ages. To be used for self-care, at institutions or workplaces without technical experience or training. Can be used with contact lenses.

### Storage:

Store between 15°C and 30°C, for comfortable use.

Storage/transport temperature: -20°C to 40°C.

### Disposal:

Follow local regulations for safe disposal of pressurized containers.

### Precautions:

- Avoid direct contact between the nozzle and the wound/eye.
- Seek medical attention if condition persists or gets worse.
- If any serious incident occurs in relation to the device, report to the local competent authority and the manufacturer.

### Warning

- Pressurised container: May burst if heated.
- Do not pierce or burn, even after use.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not expose to temperatures exceeding 50°C/122°F.
- Protect from sunlight.
- Keep out of reach of children.

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