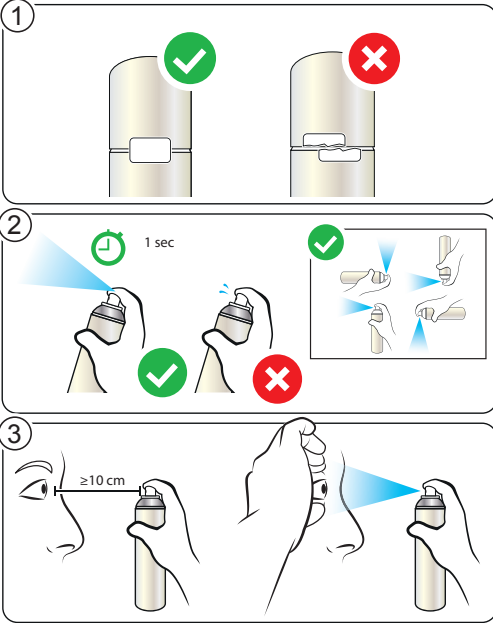
First Aid Eye Wash

TD-1451, Aurena Eye Wash

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(en) Intended use:
Sterile isotonic saline solution for rinsing/cleansing of the eye as first aid treatment.
Target user: For all ages. To be used in institutions and workplaces without technical experiences or training. Can be used with contact lenses.

Storage:
Store between 15°C and 30°C, for comfortable use.
Storage/transport temperature: -20°C to 40°C.

Disposal:
Follow local regulations for safe disposal of pressurized containers.

Precautions:

- Seek medical attention if condition persists or gets worse.
- If any serious incident occurs in relation to the device, report to the local competent authority and the manufacturer.

Warning:

- Pressurised container. May burst if heated.
- Do not pierce or burn, even after use.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not expose to temperatures exceeding 50°C/122°F.
- Protect from sunlight.
- Keep out of reach of children.

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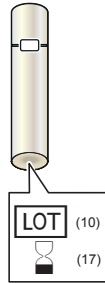
405

+ First Aid Eye Wash

STERILE SALINE EYE WASH
Sodium Chloride Solution (0.9% NaCl)

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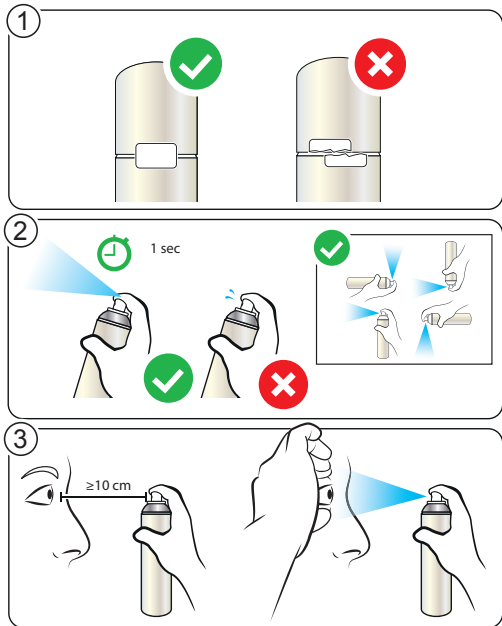
(en) Intended use:
Sterile isotonic saline solution for rinsing/cleansing of the eye as first aid treatment.
Target user: For all ages. To be used in institutions and workplaces without technical experiences or training. Can be used with contact lenses.

Storage:
Store between 15°C and 30°C, for comfortable use.
Storage/transport temperature: -20°C to 40°C.

Disposal:
Follow local regulations for safe disposal of pressurized containers.

Precautions:

- Seek medical attention if condition persists or gets worse.
- If any serious incident occurs in relation to the device, report to the local competent authority and the manufacturer.

Warning:

- Pressurised container: May burst if heated.
- Do not pierce or burn, even after use.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not expose to temperatures exceeding 50°C/122°F.
- Protect from sunlight.
- Keep out of reach of children.

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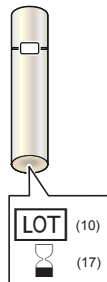
405

+ First Aid Eye Wash

STERILE SALINE EYE WASH
Phosphate Buffered Sodium Chloride
Solution (0.9% NaCl)

STERIL ÖGONDUSCH
Fosfatbuffrad Koksaltlösning (0,9% NaCl)

Distributör/Distributör:
XXXX

(01) XXXXXXXXXXXXXC

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Aurena Laboratories AB