

XX Burn Gel is a water-based gel for cooling and soothing after heat exposure where burns blanch with pressure. The gel works by forming a transparent protective "film" as a temporary protecting barrier that cools and soothes the skin thereby reducing pain. The gel is easy to apply, water soluble and is easily washed off. The transparent film enables inspection of the injury without removing the gel.

**Intended use**  
Water-based gel for first aid treatment of minor burns, scalds, sunburn and UV-radiation injured skin.

**Target user**  
Can be used for all ages. The device is to be used in self-care (lay-persons), institutions and workplaces. Can be used without technical experience or training.

**Directions**  
Apply the gel abundantly at a close distance on the burn immediately, if needed apply more after approximately 15 minutes. The gel can easily be washed or wiped off when the burn feels better or after 30-60 minutes.

**Composition**  
Purified water, Carbomer, Preservatives, Aloe Vera, Sodium Hydroxide.

**Storage**  
Store at room temperature, between 15°C (59°F) and 30°C (86°F) with a shelf life of 3 years.  
Storage/Transport temperature: -20°C (-4°F) to 30°C (86°F).

**Disposal**  
Follow local regulations for safe disposal of pressurized containers.

**Precautions**

- Seek medical help if your condition lasts or gets worse or if your burn does not blanch with pressure.
- If any serious incident occurs in relation to the device, report to the local competent authority and the manufacturer.

**Contraindications**

- Do not use on deeper than superficial burns, i.e., do not use on non-blanching burns or other types of burns such as chemical burns, phototoxic reactions, drug induced skin reactions or skin damages induced by electric shock.

**Warning**  
Pressurised container: May burst if heated.

- Do not pierce or burn, even after use.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not expose to temperatures exceeding 50°C/122°F.
- Protect from sunlight.
- Keep out of reach of children.

**Distributor:** XXXX

**MD** 175

**REF** 2004-1

**CE**

Aurena Laboratories AB  
Fjärilsvägen 22  
SE-653 50 Karlstad, Sweden

2862

V4, 2022-11-29AP

-100 ML -  
**BURN  
GEL**  
FIRST AID  
BURN GEL  
SOOTHES, COOLS  
AND RELIEVES PAIN