

TD-1873, Aurena Adhesive Remover

- 50 ML -

ADHESIVE REMOVER SPRAY

ADHESIVE REMOVER SPRAY

XX Adhesive Remover gently and painlessly removes adhesive products, such as stoma pouches and wound dressings.

Intended use
To prevent skin stripping and to ease removal of medical adhesives from the skin surface.

Target user
Can be used for all ages.

Directions
1. Spray around the edge of the adhesive and wait for a few seconds.
2. Peel adhesive away from the skin using additional spray to ease removal if necessary.

The spray dries very quickly, once dried it will not affect further appliances.

Composition
100% HMDSO.

Storage
Store between 15°C (59°F) and 30°C (86°F) for comfortable use.
Storage/Transport temperature: -20°C (-4°F) to 40°C (104°F).

Disposal
Follow local regulations for safe disposal of contents/pressurized containers.

REF 2005-1 **MD** 89

Distributor: XXXX **3**

 (01) XXXXXXXXXXXXX

LOT (10) See bottom of can

 (17)

 **2862**

Danger
Extremely flammable aerosol.
Pressurised container:
May burst if heated.

Do not pierce or burn, even after use.
Do not spray on an open flame or other ignition source.
Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
Do not expose to temperatures exceeding 50°C/122°F.
Protect from sunlight.
Keep out of reach of children.

Aurena Laboratories AB
Fjärnviksvägen 22
SE-653 50 Karlstad, Sweden

V4. 2024-07-05AP