

# SPARKLING NASAL SPRAY

ISOTONIC

XX Nasal Spray is a non-habit forming preservative-free nasal irrigation spray intended to clear and moisturize the nasal mucosa/cavity. It is to be used when the nasal passage is obstructed by nasal secretions or by other matter as blood clots or crusts. It helps to clear the nasal mucosa from particulate matter as dust or pollen and allergens and irritants. The device can be used to clear the nasal cavity and the mucosa prior to the administration of medicinal products. Can be used as frequent as needed.

## Intended use

**Intended use**  
To clear and moisturize the nasal mucosa/cavity.

**Target user**

**Can be used on children (from 2 years) and adults. Can be used in healthy individuals, in connection with irritant exposure, as well in the population with medical conditions such as allergies, acute and chronic respiratory upper tract infections, and after nasal surgery.**

### Directions

**Directions**  
Hold the can at any angle and gently insert the nozzle into each nostril, press 1-2 seconds. Clean the nozzle in soapy water and rinse with warm water after each use.

### Composition

XX


## Storage

**Storage**  
Store between 15°C (59°F) and 30°C (86°F) for comfortable use.  
Storage/transport temperature: -20°C (-4°F) to 40°C (104°F).

3 

STERILE	R	REF
---------	---	-----

 2016-X

 Aurena Laboratories AB  
Fjärrviksvägen 22  
SE-653 50 Karlstad, Sweden



2862



### Disposal

**Disposal**  
Follow local regulations for safe disposal of pressurized containers.

### Precautions

- The use of this device by more than one person may spread infections.
- Seek medical attention if condition persists or gets worse.
- If any serious incident occurs in relation to the device, report to the local competent authority and the manufacturer.

### Warning

**Warning**  
Pressurised container: May burst if heated.

- Do not pierce or burn, even after use.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- Do not expose to temperatures exceeding 50°C/122°F.
- Protect from sunlight.
- Keep out of reach of children.

Distributor:



(01) XXXXXXXXXXXXXC

**LOT** (10) See bottom of can  
 (17)

V2. 2022-08-25AP

- 100 ML -

# SPARKLING NASAL SPRAY

- ISOTONIC -



**SPARKLING  
NASAL SPRAY**

**HYPERTONIC**

- 100 ML -

**SPARKLING  
NASAL SPRAY**

**- HYPERTONIC -**

# SPARKLING NASAL SPRAY

## HYPERTONIC

**XX Nasal Spray** is a non-habit forming preservative-free nasal irrigation spray intended to clear and moisturize the nasal mucosa/cavity. It is to be used when the nasal passage is obstructed by nasal secretions or by other matter as blood clots or crusts and decongests the nasal cavity. It helps to clear the nasal mucosa from particulate matter as dust or pollen and allergens and irritants. The device can be used to clear the nasal cavity and the mucosa prior to the administration of medicinal products. Can be used as frequent as needed.

**Intended use**  
To clear and moisturize the nasal mucosa/cavity.

**Target user**  
Can be used on children (from 2 years) and adults. Can be used in healthy individuals, in connection with irritant exposure, as well in the population with medical conditions such as allergies, acute and chronic respiratory upper tract infections, and after nasal surgery.

**Directions**  
Hold the can at any angle and gently insert the nozzle into each nostril, press 1-2 seconds. Clean the nozzle in soapy water and rinse with warm water after each use.

**Composition**  
XX

**Storage**  
Store between 15°C (59°F) and 30°C (86°F) for comfortable use.  
Storage/transport temperature: -20°C (-4°F) to 40°C (104°F).

**Disposal**  
Follow local regulations for safe disposal of pressurized containers.

**Precautions**  
· The use of this device by more than one person may spread infections.  
· Seek medical attention if condition persists or gets worse.  
If any serious incident occurs in relation to the device, report to the local competent authority and the manufacturer.

**Warning**  
Pressurised container: May burst if heated.  
· Do not pierce or burn, even after use.  
· Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.  
· Do not expose to temperatures exceeding 50°C/122°F.  
· Protect from sunlight.  
· Keep out of reach of children.

**Distributor:** XXXX

(01) XXXXXXXXXX

**LOT** (10)

(17)

See bottom of can

V2\_2022-12-19AP